



Single-Action Circumcision Device



See Instructions For Use.



Single Use

STERILE R

LATEX FREE

DEHP FREE

Made in USA

P/N 054-0045 Rev. D

OPEN
HERE

Indications:

- This device is used to grasp, crush and cut the penile foreskin on full term, newborn (up to 10 days) male infants.

Contraindications:

- Hypospadias, epispadias, or megaurethra
- Ambiguous genitalia (inability to determine phenotype of child)
- Family history of bleeding problems
- Unusual appearing genitalia
- Severe illness
- Child has not had at least 1 void since birth
- The ring of the probe cannot be easily inserted and/or the foreskin appropriately aligned on the top of the ring.

Warnings:

- This device must never be used if component parts are damaged or missing. A thorough inspection of the device should be made prior to its use to ensure it has not sustained any damage.
- Possible eye hazard – care should be taken to avoid injury by keeping the pointed end of the probe away from the face of the infant and any clinical personnel.
- AccuCirc kits and parts should never be mixed. Only the components that are provided in an AccuCirc kit should be used together. Only use certified AccuCirc components. Do not use this device with parts from any other device.
- Although the 1.1 cm and 1.3 cm AccuCirc devices appear to fit most infant penises, it is possible that they may not be appropriate for very small or very large penises. In order to avoid infant injury, do not use the AccuCirc device if the size appears suboptimal.
- Prior to placing the AccuCirc ring into the foreskin, perform complete adhesiolysis per standard careful technique while avoiding the frenulum.

Cautions:

- Federal law restricts this device to sale by or on the order of a physician trained to perform newborn infant circumcision.
- This device should not be used on a child less than 12 hours old or over 10 days old.



747 West 4170 South
Murray, Utah 84123 USA
Phone: (801) 268-8200
Fax: (801) 266-7373
www.clinicalinnovations.com